



STRAUMANN® TEMPORARY ABUTMENTS  
**YOUR SOLUTION  
FOR NARROW SPACES**



COMMITTED TO  
**SIMPLY DOING MORE**  
FOR DENTAL PROFESSIONALS<sup>SM</sup>

# YOUR SOLUTION FOR


## INCREASED FLEXIBILITY

- Small diameter temporary abutment for limited interdental spaces
- Both crown and bridge options are available
- For use with either screw- or cement-retained provisionals
- Consistent Emergence Profile™
- Reliable CrossFit™ Connection

Straumann bone level temporary abutments feature the patented Consistent Emergence Profiles and CrossFit connection to help provide treatment flexibility during the healing period.


## INDICATION AND INTENDED USES

Bone level temporary abutments are cleared for use with Straumann® bone level implants for temporary restorations of single crowns and bridges for up to 6 months.

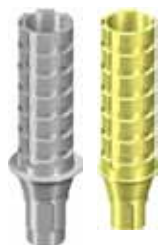

### Engaging abutments are used for

- Screw- or cement-retained temporary crowns
- Cement-retained temporary bridges

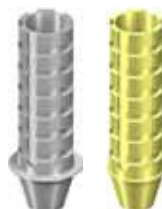

### Non-engaging abutments are used for

- Screw-retained temporary bridges

## CONTRAINDICATION


Allergies to chemical ingredients of the titanium alloy







# NARROW SPACES
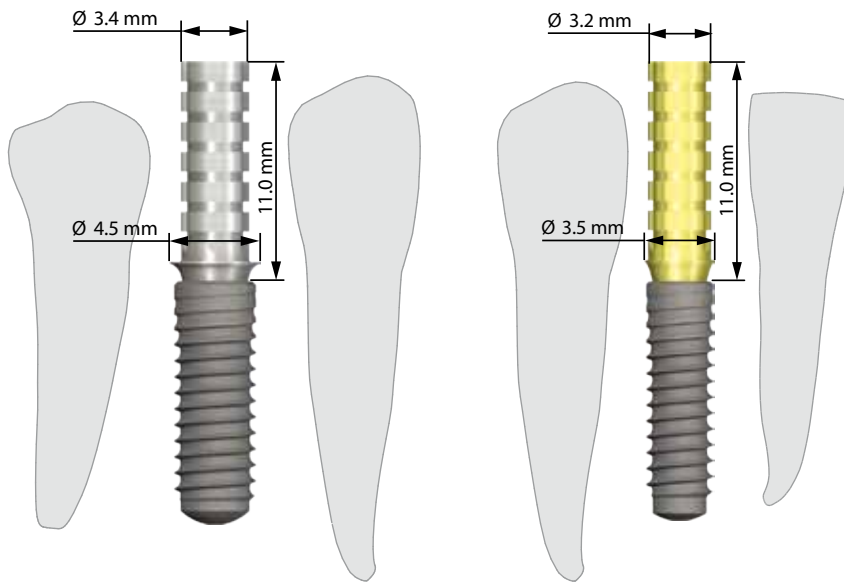
## PRODUCT INFORMATION

- The devices are provided non-sterile and are for single use only
- The abutment must be steam (134 C° / 18 min) sterilized
- Material: Titanium alloy (TAN / Ti - 6Al - 7Nb)
- Product includes screw
- Provisional restorations are to be placed out of occlusion
- Customizable up to 6 mm
- Not grindable (no diameter reduction or emergence profile customization possible)



## NC & RC Temporary Abutments\*

Art. No.	Article	Dimension
024.2371*	NC Temporary Abutment, for crown 	Ø 3.5 mm L 11.0 mm
024.2375*	NC Temporary Abutment, for bridge 	Ø 3.5 mm L 11.0 mm
024.4371**	RC Temporary Abutment, for crown 	Ø 4.5 mm L 11.0 mm
024.4375**	RC Temporary Abutment, for bridge 	Ø 4.5 mm L 11.0 mm

\* Comes with basal screw 025.2900

\*\* Comes with basal screw 025.4900

[www.straumann.us](http://www.straumann.us)



STRAUMANN GUARANTEE

### International Headquarters

Institut Straumann AG  
Peter Merian-Weg 12  
CH-4002 Basel, Switzerland  
Phone +41 (0)61 965 11 11  
Fax +41 (0)61 965 11 01

### Straumann USA

Straumann USA, LLC  
60 Minuteman Road  
Andover, MA 01810  
Phone 800/448 8168  
978/747 2500  
Fax 978/747 2490  
[www.straumannusa.com](http://www.straumannusa.com)

### Straumann Canada

Straumann Canada Limited  
3115 Harvester Road, 1st Floor  
Burlington, ON L7N 3N8  
Phone 800/363 4024  
905/319 2900  
Fax 905/319 2911  
[www.straumann.ca](http://www.straumann.ca)