


BONDING A STRAUMANN® CARES® VARIOBASE™ COPING TO STRAUMANN® VARIOBASE™ ABUTMENT

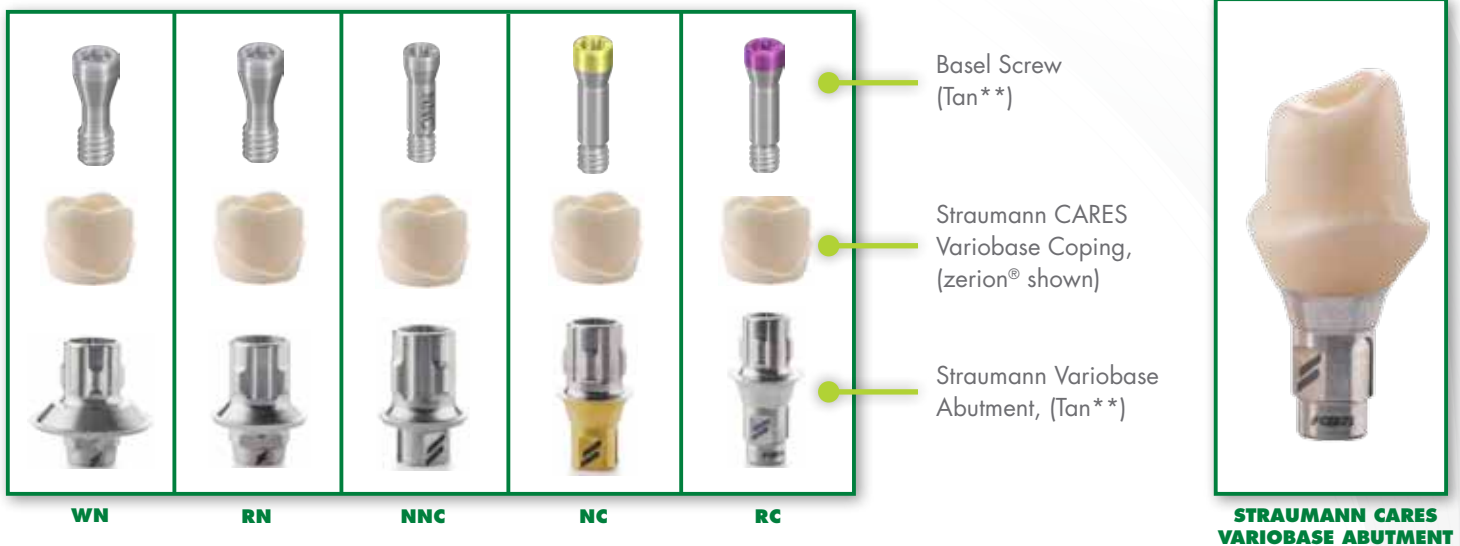

1. Seat the Straumann Variobase Abutment on the implant analog with a screw (*hand tight*). Seal the screw channel with wax.
2. Apply self-adhesive dental cement on the Variobase abutment.
3. Bond the Straumann CARES Variobase Coping to the Variobase abutment. (*Straumann recommends Panavia* F2*)
4. Immediately remove excess cement from the abutment. (*Refer to cement manufacturer's instructions*)

NOTE. For detailed instructions for bonding, handling, and seating of a variobase abutment refer to the instructions for use.



*Panavia F2.0 is a Registered Trademark of KURARAY CO., LTD. Japan
 **Titanium-aluminum-niobium alloy