



STRAUMANN® TEMPORARY ABUTMENTS
**YOUR SOLUTION
FOR NARROW SPACES**



COMMITTED TO
SIMPLY DOING MORE
FOR DENTAL PROFESSIONALSSM

YOUR SOLUTION FOR **NARROW SPACES**



INCREASED FLEXIBILITY

- Small diameter temporary abutment for limited interdental spaces
- Both crown and bridge options are available
- For use with either screw- or cement-retained provisionals
- Consistent Emergence Profile™
- Reliable CrossFit™ Connection

Straumann bone level temporary abutments feature the patented Consistent Emergence Profiles and CrossFit connection to help provide treatment flexibility during the healing period.

INDICATION AND INTENDED USES

Bone level temporary abutments are cleared for use with Straumann® bone level implants for temporary restorations of single crowns and bridges for up to 6 months.

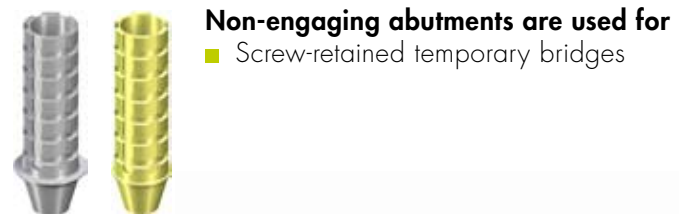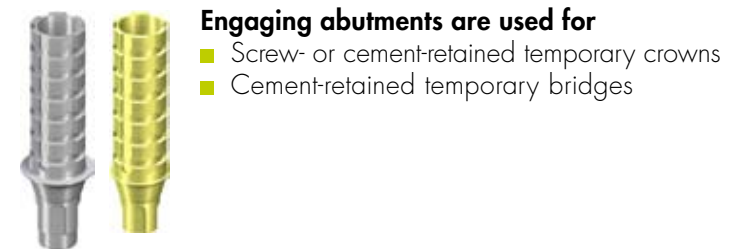


CONTRAINDICATION





Allergies to chemical ingredients of the titanium alloy

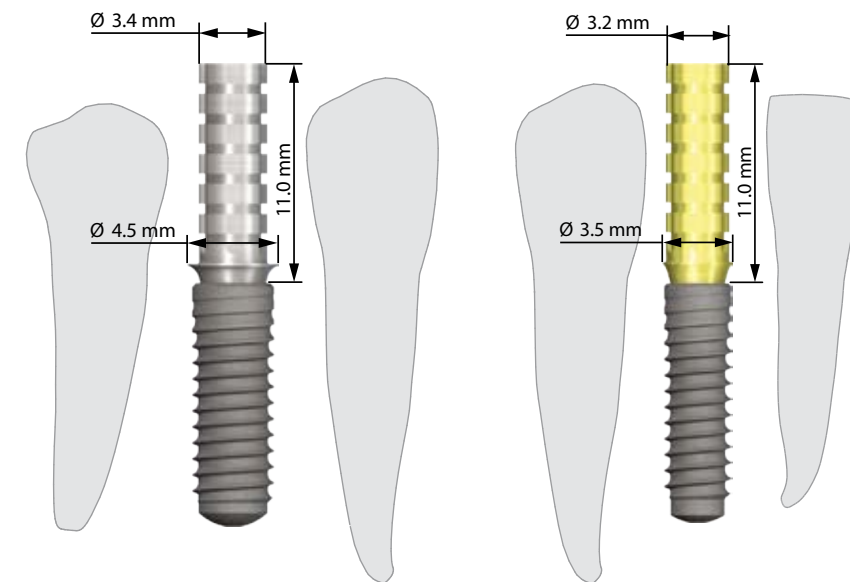PRODUCT INFORMATION

- The devices are provided non-sterile and are for single use only
- The abutment must be steam (134 C° / 18 min) sterilized
- Material: Titanium alloy (TAN / Ti - 6Al - 7Nb)
- Product includes screw
- Provisional restorations are to be placed out of occlusion
- Customizable up to 6 mm
- Not grindable (no diameter reduction or emergence profile customization possible)



NC & RC Temporary Abutments*

Art. No.	Article	Dimension
024.2371 *	NC Temporary Abutment, for crown 	Ø 3.5 mm L 11.0 mm
024.2375 *	NC Temporary Abutment, for bridge 	Ø 3.5 mm L 11.0 mm
024.4371 **	RC Temporary Abutment, for crown 	Ø 4.5 mm L 11.0 mm
024.4375 **	RC Temporary Abutment, for bridge 	Ø 4.5 mm L 11.0 mm

* Comes with basal screw 025.2900

** Comes with basal screw 025.4900

www.straumann.us



STRAUMANN GUARANTEE

International Headquarters

Institut Straumann AG
Peter Merian-Weg 12
CH-4002 Basel, Switzerland
Phone +41 (0)61 965 11 11
Fax +41 (0)61 965 11 01

Straumann USA

Straumann USA, LLC
60 Minuteman Road
Andover, MA 01810
Phone 800/448 8168
978/747 2500
Fax 978/747 2490
www.straumannusa.com

Straumann Canada

Straumann Canada Limited
3115 Harvester Road, 1st Floor
Burlington, ON L7N 3N8
Phone 800/363 4024
905/319 2900
Fax 905/319 2911
www.straumann.ca